

ASSEMBLY INSTRUCTIONS

HALF MOBILE BARRIER (MB-HALF)

1. Slip channels (item 2) onto each side of panel (item 3), with holes closest to bottom.
2. Attach handle (item 1) to channels (item 2) and passing through panel (item 3) with (2) M6-35 Bolts, (4) 1/4" washers (one washer on each side of the channel) and (2) M6 acorn nuts "hand tight"
3. Slip triangle legs (item 4) over channels (item 2) with slotted hole on the same side as handle (item 1).
4. Align holes and insert (4) M6-40 bolts and (4) 1/4" washers through triangle legs (item 4), channels (item 2) and panel (item 3) with (4) M6 acorn nuts and (8) 1/4" washers (one washer on each side) "hand tight"
5. Attach all (4) caster wheels (item 6) with 3/8" nylock nuts and 3/8" washers (one washer on each side of leg) and tighten.
6. Slide leaded acrylic panel (item 5) into channels (item 2) and down onto panel (item 3).
7. Grasp channels (item 2) at both sides of panel (item 3) and compress unit by squeezing inward to panel (item 3). Now tighten M6-40 and M6-35 bolts and acorn nuts using allen wrench and 10mm wrench (not included).

Your unit is now ready for use.

ITEM #	QTY.	PARTS LIST
1	1	Handle
2	2	Side Channels
3	1	Panel
4	2	Triangle Legs
5	1	Leaded Acrylic Panel
6	4	Caster Wheels
7	4	3/8" Nylock Lock Nuts
8	6	M6 Acorn Cap Nut
9	8	3/8" Flat Washer
10	12	1/4" Flat Washer
11	4	M6-40 Bolts
12	2	M6-35 Bolts
13	1	Allen Wrench

